

Q&A

Q: Does the patient need to be shaved in the area of application?

A: No, the patient does not need to be shaved in the area.

Q: How frequently should the UriCap be changed ?

A: The uricap is intended for up to 24 hours use and it should be changed daily.

Q: Can the uricap be reused?

A: No, the UriCap is a single use device, and should be discarded in the medical waste disposal after use.

Q: Is it safe and clean to use the UriCap female?

A: Yes, the UriCap Female is a clean product, made from high grade medical Silicone and has the necessary regulatory approvals in each country it is sold in.

Q: Can a patient walk around with the device?

A: Yes, walking around can be done safely after the device have been in place for a few hours. To be sure dislocation has not happened, please check position after walking.

Q: Can the device be used while wearing underwear?

A: Yes , you may use loose underwear, or pants. See the tube postion explanation in section F.

Q: What should be done if urine flow has stopped?

A: Firstly check if there is a leak, then check if there is uninterrupted flow in the urine tube. Correct any tube twisting or kinking that may have occurred.

Q: What should be done if a leakage* is observed?

A: First, check that the UriCap is in full contact with the skin around the urethra, then verify the following steps:

- The device is completely covered by labia.
- The UriCap device and the extension tube are firmly connected.
- The Extension Tube and the urine bag are firmly connected.

* It is recommended to monitor leakage and tube positioning each time patient care and wet check is performed .

Q: What should be done in order to prevent, accidentally pulling out the UriCap?

A: Use medical tape to secure the extension tube to the inner thigh. Don't forget to leave some slack so the tube can move. Move the urine bag to the correct position before turning the patient. Make sure mobilization of the patient is done carefully.

Q: If a patient complains about discomfort some time after the installation, what should be done?

A:

- Check if the device is installed correctly as described in the instructions. Make sure it is covered by both labias and didn't slide into the vaginal opening, or dislocate from the area surrounding the urethra. Correct the position.
- Check if there is redness, irritation, a wound, bleeding or secretion in the area of installation. If so, remove the device , check the area properly and consult the physician.
- Please fill out the Adverse event form.