For more information and the application movie:  
[www.tillacare.com/training](http://www.tillacare.com/training)

Temperature range: 50°F / 10°C to 86°F / 30°C

**TillaCare Ltd.**  
PO box 2093, Kanion Lev Ha'ir, 39 Herzl St., Tirat Carmel 3912002, Israel  
**Visiting Address** 2 Etgar St., Carmel building, Tirat Carmel 3903213, Israel  
Tel/Fax +972-7-75508737  
[www.TillaCare.com](http://www.TillaCare.com)

MedNet EC-REP GmbH  
Borkstrasse 10,  
48163 Muenster, Germany

IFU00003-C JUN 2021

**ENG Instructions For Use**

**General Information**  
**UriCap Female** is a non-invasive, single-use pouch, which is attached to the periurethral area and which can be connected to a urinary drainage container such as a bag or bottle. The device simplifies urine collection and enables easy monitoring of the urine volume. UriCap is designed to prevent exposure of urine to the skin, preventing wetness, thus protecting the skin. UriCap Female is designed for adult patients that suffer from continuous urinary incontinence and should be changed daily.

**Indications For Use**  
The intended use of the UriCap Female, is to be a conduit for urine collection, via a pouch and tubing, into a gravity drainage collection bottle or bag. The device is intended to be used on women age 18 or older, by a trained person, after consultation with a physician or registered nurse.

**The Product**  
UriCap Female comes in two parts. The Silicone Pouch and the Extension Tube. They are packed in separate packages. Before fitting the Silicone Pouch, open both packages and assemble the parts into one piece.

A - Supporter  
B - Pouch (silicone)  
C - Holder  
D - Device Tube  
E - Connector  
F - Extension Tube

**Assembly**

a. Attach the Connector (E) to the Device Tube (D).  
b. Attach the Extension Tube (F) to a standard urine bag.

- Make sure the Device Tube (D) and the Connector (E) are well connected.
- Make sure the Extension Tube (F) and urine bag are well connected.
- Make sure there are no kinks in the Device Tube (D) or the Extension Tube (F).

**Application**

- Start with cleaning the area of vagina vulvae. Using soap and water or cleansing wipes cleanse the vaginal area thoroughly from top to bottom, leave any remaining moisture on skin.
- Place a small amount of water-based lubricant around the connecting surface of the device. Recommended holding position.  
**Notice:** when held horizontally the UriCap has an upper/shorter lip and a lower/longer lip. The lower/longer lip is the bottom lip.
- With the patient lying down and legs spread apart, again carefully spread open the inner and outer labia wide enough with your non dominant hand so the pouch can be placed all the way in. Use your dominant hand to fit the supporter (A) (the lower lip) at the vaginal opening by holding the device as illustrated in fig 3.
- Using your thumb push upward. Gently push in the device to ensure that there is full contact with the skin and hold for 5-10 seconds to ensure adherence.  
**Notice:** Make sure that the inner labia is wide open and not caught under the device during application.
- Close the inner and outer labia around the Silicone Pouch, only then release your dominant hand.

- The outer labia should completely cover and conceal the UriCap Holder, if not, there may be leakage.
- Secure the Extension Tube to the inner thigh using a securement device.

- Connect the securement device around the urine bag's connection to the extension tube.
- Position user with legs spread widely apart and localize appropriate position with enough slack to the tube so that the patient can move the legs without pulling out the device.
- Clean the area with alcohol pad, allow to dry, and attach the securement device.

The fitting is now completed. Change the securement device according to facility policy, rotate between right and left leg with each change of the securement device. The patient should remain in a lying position for at least an hour before position change. If the patient uses adult briefs due to fecal incontinence, the tube must pass through the lower end of the adult briefs.

**Always remember**

- Change the location of the urine bag before changing the patient's position.
- The urine bag should always be positioned below the urethra level and beside the bed / chair.
- Ensure that there is enough slack in the tubing to avoid pulling on the urine bag while moving.
- When lying or sitting down the tubing of the urine drainage bag should be placed under / behind the bend of the knee to avoid pressure of the tube on the leg.

**Removal**

8&9. Spread apart the outer and inner labia. Gently peel from the top of the device downward and remove, or use thumb and index finger to locate the device, then squeeze to release the seal.

**Caution:** Do not pull the device straight out. It may cause discomfort and damage to the tissue.

**Replacement**  
To replace the UriCap Female with a new device without replacing the Extension Tube or urine bag, disconnect the Extension Tube from the UriCap Female, connect a new UriCap Female device and repeat step 1-7.

**Cautionary notes**

- Do not use oil or silicon-based lubricant
- Please keep away from children.

**Possible undesirable side effects**  
The following undesirable side effects may occur and should be monitored: pain, itch, rash, irritation, chafe, pressure sore. If any of these side effects occurs, please discontinue using the device and consult a doctor. Before reapplying the device make sure any side effects have been fully resolved.

**Contraindications**

- UriCap is not to be used on patients sensitive or allergic to silicone.
- If the patient has vaginal discharge, itch, inflammation or skin condition, consult a doctor prior to using the device.
- The device is not suitable for patients with urinary retention.
- The device is not suitable for patients with pelvic prolapse (stage 2 and up) due to changes in vaginal anatomy

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