




Quick Reference Guide


by TillaCare Ltd.

Assembly


Place the assembled urine bag and tubing connector in the middle of the white stripe (Figure A).

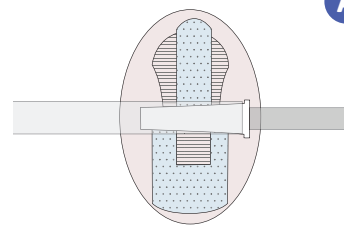

Grasp both tabs and feed single tab through opening in the double tab (Figure B).

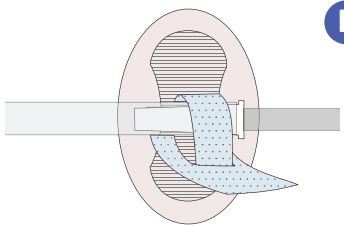


While grasping both tabs, pull them in opposite directions until the device tubing is snugly captured and push each to down on the mating Velcro base surface (Figure C).

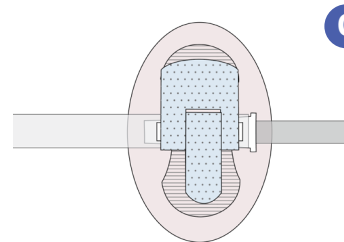


Application


Apply the urine collection device

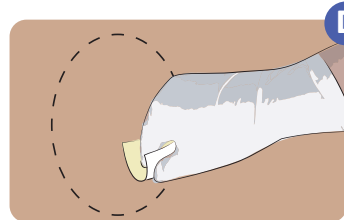

Cleanse application area (Figure E) with alcohol prep pad. Allow area to completely dry.

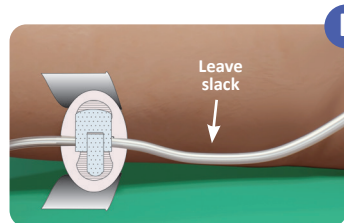


Leave ample slack in the tubing to avoid pulling. Peel off one side of the paper backing and press to skin. Repeat on the other side (Figure E+F) making certain all edges are adhered firmly to skin.

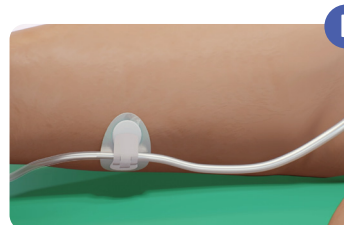

Urine bag replacement

The securement device tabs may be opened for Urine bag replacement (Figure G).

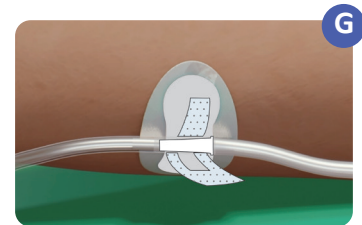

Removal

Pull both tabs away from the Velcro.

Slowly peel the Stabilization Device off the skin.

If desired an alcohol prep pad may be used to help loosen the adhesive

Tip

Replace the securement device every 3 days, it is recommended to switch legs after each application.



Always review the Instructions for Use (IFU) including indications, contraindications, warnings, precautions. Follow facility policy when using the product.

TillaCare Ltd.
PO box 2093, Kanion Lev Ha'ir, 39 Herzl St.,
Tirat Carmel 3912002, Israel
Visiting Address: 2 Etgar St., Carmel building,
Tirat Carmel 3903213, Israel
E-mail: USsales@tillacare.com

Copyright © 2021 TillaCare Ltd. All rights Reserved.
DOC00031 REV.A JAN 2022